



Specifications:

Maximum capacity: 350 lbs
15 feet of strong, stainless steel cable

Worry free

If the wheelchair is not going up the ramp straight, a breaker prevents you from being pulled over the side of the ramp. This system resets in only a few seconds, allowing you to reposition yourself. All the while, the **POWER PULL** cable is still holding you in place on the ramp.

Safe exit

When exiting the vehicle, the **POWER PULL** provides steady tension to help you back down the ramp. In the event that your wheelchair stops its descent because it is hooked on the ramp, the unique clutch system disengages the motor even if the 'OUT' button is being held. This means that when the wheelchair is straightened out, there is no slack in the cable, therefore, no uncontrolled rolling back down the ramp. Once the wheelchair is repositioned, the **POWER PULL** resumes normal operation.

Installation

The **POWER PULL** is ideal for lowered floor minivans. It takes up little space in the vehicle and therefore is very versatile.



We take our customers' **safety** very seriously. We are equipped with computerized seat and seat belt testing equipment so we can test all of our products in or out of a vehicle. This allows us to stay on the leading edge. Our mission is to supply you with the safest, quality mobility equipment available in today's market.

Dealer :



145, Damase-Breton
Saint-Lambert-de-Lauzon
Quebec, Canada
G0S 2W0

POWER PULL

Wheelchair & passenger ramp assist system



www.adapt-solutions.ca

POWER PULL

Wheelchair & passenger ramp assist system

Functions:

- Roll up beside your vehicle
- Take hold of the **POWER PULL** strap
- Position yourself in front of the ramp
- Place the hooks on the chair
- Press and hold the 'IN' button
- The **POWER PULL** takes you up the ramp
- Can be used to exit the vehicle

Independent use

With the ease of a wireless remote, the **POWER PULL** can be reached from outside the vehicle, making it a great solution for drivers in wheelchairs!

Easy to use

With an innovative clutch system, the **POWER PULL** holds tension under all circumstances. It is equipped with a retractable, auto locking system. Therefore it cannot unravel or get tangled up. It always pulls in any slack.

No need to wait

The 'RELEASE' button removes all tension on the cable. This way you don't need to wait for the cable to unroll. Simply pull the hooks to where you need them. The 'RELEASE' button also has a safety on it. It will not release if the **POWER PULL** is in use.

Safety

If buttons are accidentally released while going up or down the ramp, your wheelchair stays in place and you will not roll back down the ramp.

